

MAXIGUARD

FACE SHIELD



PRODUCT DESCRIPTION

- MAXI-GUARD Face Shield

TECHNICAL SPECIFICATIONS

- Assembled weight 46g
- 0.4mm/400micron PET lens 300mm wide x 220mm deep
- 25mm black elastic headband
- 3 slot side fastening and adjustment for headband
- 25mm x 30mm closed-cell rigid foam sweatband
- Instruction & warning label

INSTRUCTIONS FOR ASSEMBLY

1. Place lens on a flat surface with the label at the top facing down.
2. Remove the backing liner from the foam strip.
3. With adhesive side facing down, firmly place the foam strip centred along the top edge of the lens. Use the back of the label as a guide.
4. Keeping the lens face down, thread the end of the headband under the side of the lens and through the 1st slot to the inside of the lens. Thread through the 3rd slot back to the front of the lens. Flip the lens over and thread through the middle slot. Repeat step 4 on the opposite side.

INSTRUCTIONS FOR USE

Place the foam strip centred on forehead. Ensure the lens extends past the chin. Pull the headband over the top and secure on the back of the head. Adjust tightness accordingly by pulling the ends of the headband. Clean with an alcohol based sanitiser and soft fibre free cloth. Take care not to scratch the lens.

WARNING

This face shield is not certified to any industrial and/or medical specifications. It is not suited for any industrial use. The sole purpose is to be used as a disposable precautionary guard for the user against particles and liquids projected from the human body. It must be used in conjunction with additional personal protective equipment such as eye and respiratory protection. If unsure of the applications and limitations, consult your safety officer or the manufacturer.